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WÖHLK CONTACT CARE Kombilösung

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WÖHLK CONTACT CARE

MULTIFUNCTIONAL SOLUTION

Cleaning, removal of protein and lipid deposits, disinfecting, inserting and storage.

For soft contact lenses.

MEDICAL DEVICE

Read the instructions and warnings carefully for proper use of the WÖHLK CONTACT CARE in combination with your contact lenses.

Precautions and instructions for use:

- Wash your hands carefully before removing the lenses from your eyes or the lens case. Dry your hands using a clean towel.
- Unscrewing the cap from the bottle will break the guarantee seal.
- Immediately after removal of the lenses, clean them with about 8 drops WÖHLK CONTACT CARE.
- Clean the lenses by carefully rubbing them with your index finger in the palm of your other hand for 30 seconds.
- Rinse the lenses with WÖHLK CONTACT CARE.

the lens case filled with fresh WÖHLK CONTACT CARE for at least 4 consecutive hours.

- The lenses moisturised by WÖHLK CONTACT CARE can then be inserted.
- Rinse the lens case with WÖHLK CONTACT CARE and allow the lens case to dry in the air.
- Do not touch the dropper tip of the bottle as this may contaminate the solution.
- Close the bottle tightly immediately after use.

Other instructions:

- After storage in excess of 7 days, lenses must be cleaned and disinfected again before wear.
- Replace your lens case frequently to prevent infection due to contamination.

Warnings:

- Do not use if guarantee seal is damaged.
- Do not mix or use WÖHLK CONTACT CARE with other contact lens fluids, unless it is indicated in the instructions for use.
- Always replace any remaining solution in the lens-case with fresh solution.
- Do not use after the expiry date.
- Do not change your contact lens system without consulting your doctor / contact lens specialist.
- Never use tap water as an alternative for this contact lens fluid.

Do not use:

- If you are prescribed eye medication, consult your ophthalmologist or contact lens specialist before using the medication in combination with contact lenses.
- In case of eye irritation, discontinue use and consult your doctor/contact lens specialist.

Store:

- Keep out of the reach of children.
- Store between 4°C and 25°C (40°F and 77°F).
- Protect from sunlight.
- Keep the solution in its original packaging.
- Use within 4 months of opening.

Side Effects:

Do not use WÖHLK CONTACT CARE if you are allergic to one of the ingredients.

Composition:

WÖHLK CONTACT CARE is a buffered, isotonic solution containing the following active ingredients: Nemoport, Preservative: PHMB 0,0001%.

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